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
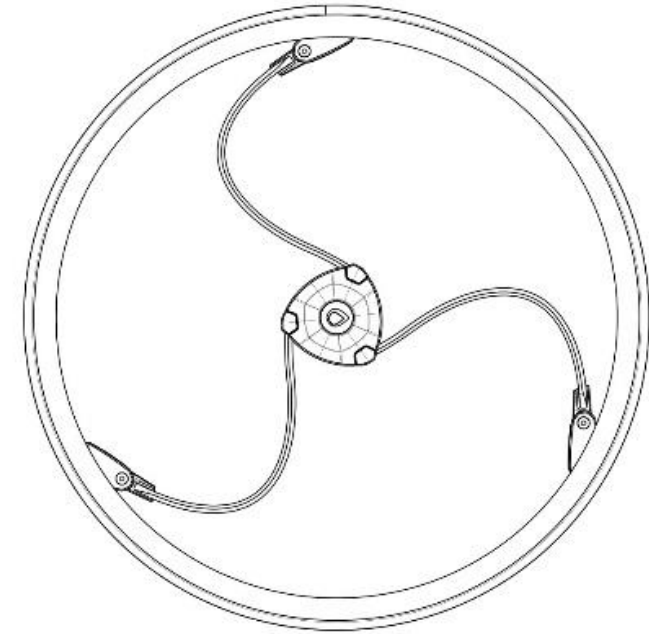
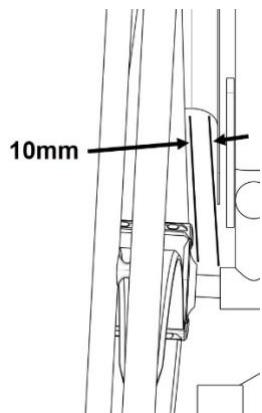
STEP 8: Rotate the wheel slowly to check that no part of the Loopwheel™ touches or rubs the frame of the chair. We recommend a minimum clearance of 10mm at all points.

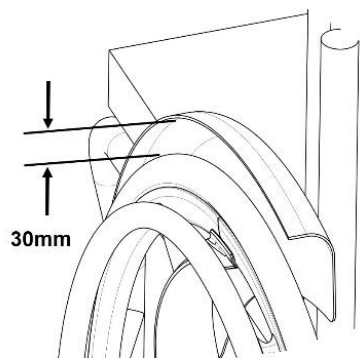In addition, there must be a space of at least 30mm above the wheel.


If the clearance is good, move to STEP 12.


If there is a clash or not enough clearance, move to STEP 9.

STEP 9: Remove the Loopwheel from the chair, place one of the steel spacers onto the axle from the INNER side of the wheel. This will move the Loopwheel further away

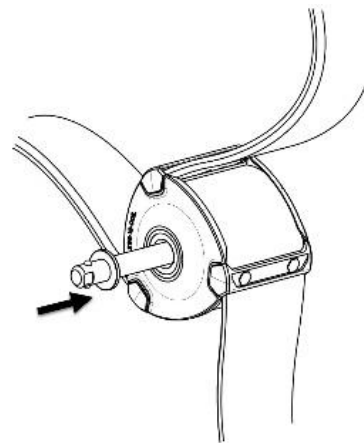from the wheelchair frame. Repeat Steps 8 and 9 up to a maximum of 3 times until there is the correct amount of clearance between the wheel and the frame of the chair.

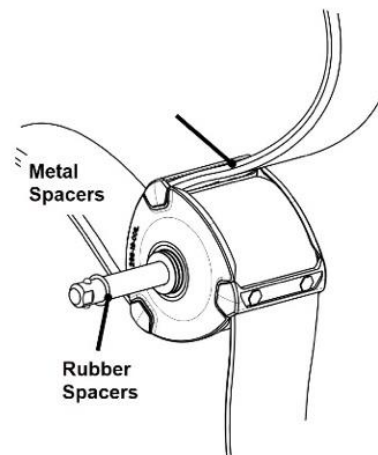STEP 10: Secure the steel spacers in place by sliding the rubber washer onto the Loopwheel™ axle.

STEP 11: Replace the wheels onto the chair.

STEP 12: Check that the removable axles are fully engaged in the axle housing of the wheelchair.

STEP 13: Check that the brakes still engage properly with the tyre. If they do not, seek advice from your wheelchair supplier or therapist on re-positioning the brakes.

6.5 Fitting and removing Loopwheels™ in everyday use

⚠ WARNING

– Always ensure that the removable axles are fully engaged whenever you fit a wheel.

Loopwheels™ vanteiden asentaminen

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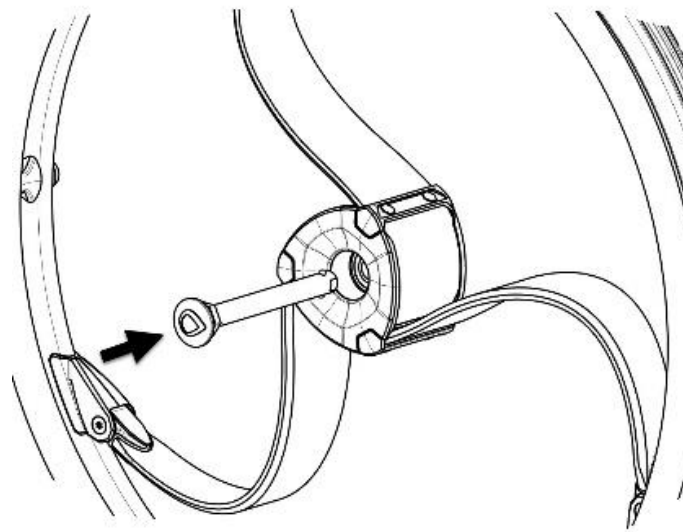
STEP 2. With one hand, hold the wheelchair upright.

STEP 3. With the other, hold the wheel around the wheel hub. Choose the wheel with an R in the product identification number for the right side, and L for the left side.

STEP 4. Using your thumb, press the removable axle button and hold it down.

STEP 5. Push the axle into the wheelchair axle housing up to the stop, again checking the wheel with an R is on the right and the L for the left side.

STEP 6. Release the removable axle button and make sure that the wheel is secure.

Removing Loopwheels™

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VAIHE 2. With one hand, hold the wheelchair upright.

VAIHE 3. With the other, hold the wheel around the wheel hub.

VAIHE 4. Using your thumb, press the removable axle button and hold it down.

VAIHE 5. Pull the axle out of the wheelchair axle housing

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⚠️ WARNING

Risk of accidents: Uneven tyre pressure can have a huge effect on handling. The pressure should be the same in both tyres.

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⚠️ CAUTION

Risk of crushing: There may be a very small gap between the loopwheel and the mudguard with the risk that you could trap your fingers.

- Ensure that you always propel your wheelchair using the handrims only.

7.2 Braking during use

Whilst you are moving, you brake by transferring force to the push rim with your hands.

⚠️ CAUTION

Risk of burning your hands If you brake for a long time, a lot of frictional heat is produced at the push rims

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1. Hold the handrims and press evenly with both hands until the wheelchair stops.

7.3 Driving and steering a wheelchair fitted with Loopwheels™

You drive and steer a wheelchair using the push rims. Fitting Loopwheels™ to your chair changes the feel of the wheelchair compared with spoked wheels. Before driving without an assistant, accustom yourself to how the chair feels and behaves.

⚠️ WARNING

Risk of falling out of the wheelchair! When using Loopwheels™ on a wheelchair that you are used to using with rigid spoked wheels, the centre of gravity may change.

- Find your wheelchair's tipping point and use Loopwheels™ initially with an assistant
- Adjust your driving style accordingly.

⚠️ WARNING

Use of a wheelchair on rough terrain may be difficult after you are used to driving on flat surfaces

- adjust driving style accordingly

8. Kuljetus

8.1 Safety Information

⚠️ WARNING

NEVER transport an occupied wheelchair fitted with loopwheels™ in a vehicle. It has not been designed for this purpose. ALWAYS transfer the wheelchair user to the vehicle seat with the seatbelt on.

⚠️ WARNING

Injury or damage may occur from loopwheels™ or other wheelchair components or accessories loosened during a collision or sudden stop.

- Ensure your wheelchair, Loopwheels™ and accessories are securely stored in the vehicle.

8.2 Transporting a wheelchair fitted with Loopwheels™ in a vehicle

Remove Loopwheels™ from your wheelchair to transport it more easily in a vehicle, by following the instructions in section 5 for removing and fitting Loopwheels™.

9. Kunnossapito

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⚠️ WARNING

Some materials deteriorate naturally over time. Wheelchair manufacturers recommend that your wheelchair should be checked by a specialist dealer at least

once a year or if it has not been used for a long period. We recommend your Loopwheels™ are included in this annual check.

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To ensure safe and reliable operation, carry out the following checks and maintenance regularly, or have the checks carried out by another person.

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1. Examine your wheels for loose parts, cracks or other defects.
2. If you find anything, have your wheels checked immediately by your loopwheels™ dealer or contact info@loopwheels.com.

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1. Examine the springs for any signs of wear, cracks, looseness or other defects.
2. Listen for any clicking sound from the wheels as they turn.
3. If you find anything, have your wheelchair checked immediately by your loopwheels™ dealer or contact info@loopwheels.com.

Tarkista kelausvanteet

1. Examine the pushrims for any signs of wear, cracks, looseness or other defects.
2. Listen for any sound coming from between the push rim and the rim: a nut attaching the push rim to the wheel rim may have worked loose. Tighten all nuts so the push rim is fitted snugly to the wheel. But don't overtighten and break the nut!

⚠️ IMPORTANT

Tarkista rengaspaineet

1. Tarkista renkaiden paine – katso 5.1.
2. Inflate the tyres to the required pressure.
3. Check the tyre tread at the same time.
4. Jos tarpeellista, vaihda renkaat.

Check that Loopwheels™ are seated correctly

1. Pull on the Loopwheel™ to check that the removable axle is seated correctly. The wheel should not come off.
2. If the Loopwheels™ are not engaged properly, remove any dirt. If the problem persists, have the removable axles re-fitted by a specialist dealer.

Tarkista jarrut

⚠️ IMPORTANT

The parking brakes may need to be repositioned after replacing your rear wheels with loopwheels™

1. Check that the parking brakes are positioned correctly. The brake is set correctly if the brake shoe depresses the tyre by a few millimetres when the brake is applied.
2. If you find that the setting is not correct, have the brakes correctly set by a specialist dealer.

⚠️ IMPORTANT

Loopwheels™ may sustain visibly undetectable damage as a result of a heavy collision or hard blow.

– It is essential to have your wheelchair and loopwheels™ checked by a specialist dealer after a heavy collision or hard blow.

Repairing or changing an sisäkumi

1. Remove the Loopwheel™ and release any air from the inner tube.
2. Lift one tyre wall away from the rim using a bicycle tyre lever. Do not use sharp objects such as a screwdriver which could damage the inner tube.
3. Pull the inner tube out of the tyre.
4. Repair the inner tube using a bicycle repair kit or, if necessary, replace the tube.
5. Inflate the tube slightly until it becomes round.
6. Insert the valve into the valve hole on the rim and place the tube inside the tyre (the tube should lie right round the tyre with no creases).
7. Lift the tyre wall over the edge of the rim. Start close to the valve and use a bicycle tyre lever. When doing this, check all the way round to ensure that the inner tube is not trapped between the tyre and the rim.
8. Inflate the tyre to the maximum operating pressure. Check that no air is escaping from the tyre.

9.3 Puhdistus

Regular cleaning of your Loopwheels™ will help them last well. Regular cleaning will reveal loose or worn parts and enhance the smooth operation of your wheels. To operate properly and safely, your wheels must be cared for just like any other vehicle wheels.

⚠️ IMPORTANT

Sand and seawater can damage the bearings, and steel parts can rust if the surface is damaged.

– Only expose your loopwheels™ to sand and seawater for short periods, and clean the hub after every trip to the beach.

⚠ CAUTION

- Do not use coarse abrasives, aggressive cleaning products or high pressure cleaners. Never use acids, alkalines or solvents such as acetone or cellulose thinner.
- Always use warm soapy water to clean.

⚠ IMPORTANT

- Do not use solvents or kitchen cleaners

1. Clean the metal parts with a soft, damp cloth.
2. Dry the hub carefully with a cloth after using it in the rain.
3. If the wheels are dirty, wipe off the dirt as soon as possible with a damp cloth and dry them carefully.

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Faults may arise as a result of daily use, adjustments or changing demands on the wheels. The table below shows how to identify and repair faults.

⚠ WARNING

- Contact your Loopwheels™ dealer immediately if you notice a fault with your wheels, e.g. a significant change in handling.

⚠ IMPORTANT

- Some of the actions listed must be carried out by a specialist dealer or by the manufacturer. These are indicated.

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© 2020 Jelly Products Ltd Kaikki oikeudet pidätetään. Uudelleenjulkaisu, kopiointi tai muuttaminen kokonaan tai osittain on kielletty ilman etukäteistä kirjallista lupaa Jelly Products Ltd. Nimi Loopwheels™ ja Loopwheels™ -logo ovat Jelly Products Ltd: n omistamia rekisteröityjä tavaramerkkejä.
© 2020 Jelly Products Ltd
Nimi Loopwheels™ ja Loopwheels™ logo ovat rekisteröityjä tuotemerkkejä jotka omistaa Jelly Products Ltd.
Valmistaja: Jelly Products Ltd. Unit 202 Boughton Industrial Estate, Newark, Nottinghamshire, NG22 9LD, UK.